



KatKath®

ADJUSTABLE TOMCAT URINARY CATHETER

with new **KatKlip**



- The adjustable **KatKath®** with **KatKlip**, a simple locking fixation device.
- Simple open and close movement.
- Introduces the rotating collar used on the standard **KatKath®** catheters to avoid 'twisting' injuries.
- The **KatKlip** is small, more comfortable and better tolerated.
- The locking component is a different colour to the body of the fixation device for ease of use.
- More easily sutured in place.
- To use the full 18cm length simply remove the **KatKlip** and use as a standard **KatKath**

Product Code:
AK3518K


Details:
3.5Fr up to 18cm maximum
usable length with Luer lock.



The Veterinary
Company Ltd


www.theveterinarycompany.com

Be kind to
your patients

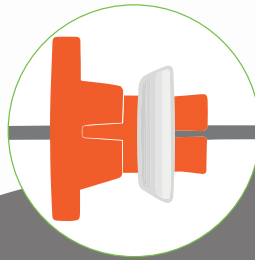

Step One

Ensure the KatKath and KatKlip are in the premeasured position.

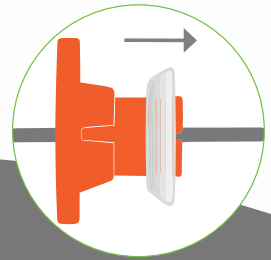

Step Two

Pull the white collar towards the hub end of the catheter to click into place.

 **KatKath**[®]

ADJUSTABLE TOMCAT URINARY CATHETER

**GREAT
LONGITUDINAL
STRENGTH**

The KatKath's increased flexibility does not diminish its ability in the effective treatment of blocked cats due to its great longitudinal strength and will not concertina even when encountering gross resistance.

**CRUSH
RESISTANT**

The KatKath[®] is almost impossible to crush and is also highly resistant to kinking. This allows placement without the use of a stylet. (Useful if flushing is required during the act of placement).

**WELL
TOLERATED**

The KatKath[®] may be left in place for hours or days, dependant on individual needs, and is extremely well tolerated by even the most difficult patients.

**KATKATH[®]
SAVES TIME
AND MONEY**

There is now no longer any need to purchase two separate catheters for flushing blocked cats and leaving a different catheter in place. The KatKath[®] is highly effective in both situations saving time, money and effort, by cutting two procedures down to one.

TO PLACE YOUR ORDER CONTACT US AT:

The Veterinary Company Ltd.

Tel: +44 (0) 1603 340 143

Fax: +44 (0) 1603 340 286

Email: sales@theveterinarycompany.com


The Veterinary
Company Ltd

www.theveterinarycompany.com